



## Safety Instructions

### How to use the device:

Voice Selector Converse including wireless earphones is a unique assistive listening device, reducing ambient noise and focusing audio in the selected direction.

Storage and operation temperatures should be between 10° and 45°C, and humidity 10% to 95%.

### How to clean the device:

The device should be cleaned with a damp cloth.

### Instructions for disposing of electronic equipment:

The electronic waste must be treated according to local regulations. Nuance Hearing Ltd. recommends recycling the device at the regional disposal of electronic equipment. Local laws must be followed.

### According to the regulations, we are required to explain the following warnings:

- Warning: Prolonged listening at high volume is not recommended and may cause irreversible damage to your hearing
- Warning: Use of accessories, components, or spare parts not supplied by Nuance Hearing may impact use and void the warranty
- The battery in Voice Selector Converse and in the wireless earphones are not replaceable and should not be replaced
- If you use other medical devices or wear implantable medical devices such as defibrillators or pacemakers and are concerned that your Voice Selector Converse might cause interference with your medical device, please contact your physician or the manufacturer of your medical device for information about the risk of disturbance.

### Standards and regulations:

Voice Selector Converse and wireless earphones meet the strictest international standards for electrical safety and electromagnetic compatibility.

