

User guide




Getting started

Voice Selector Converse and the earphones come paired, so turn them on and enjoy the conversation

① Turn on the earphones

Wear the earphones and press the power button.
You'll hear: "Power on"

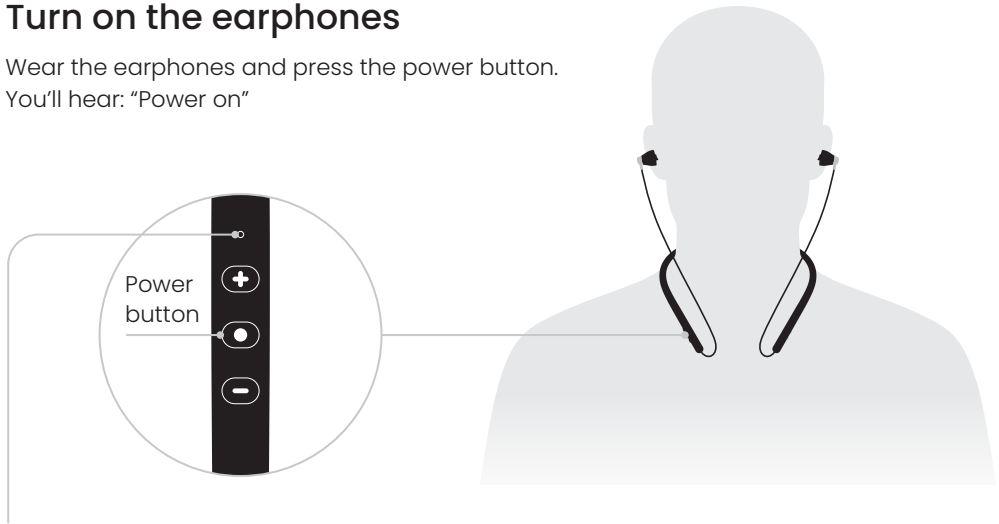


Indications:

• Flashing red
Searching for device

• Flashing blue
Device found
You'll hear: "Connected"

② Turn on Voice Selector Converse

Press the power button

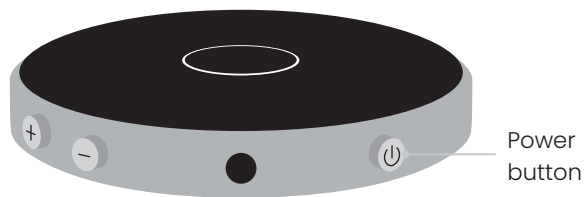

Indications:



Battery level



Searching for paired earphones




Ready for use

Live conversation modes

The device switches seamlessly between Table and Remote mic modes according to its position

Table mic


Voice Selector Converse scans the open directions and picks out the dominant speaker automatically

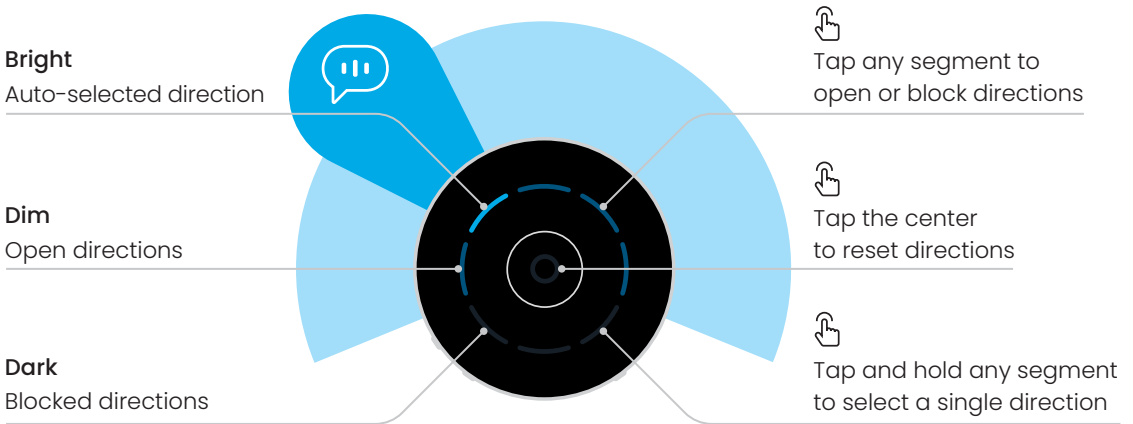

Device orientation: open segments face the group, blocked segments face you


Remote mic

Let your conversation partner wear Voice Selector Converse for optimal clarity in situations like walking, driving or cooking

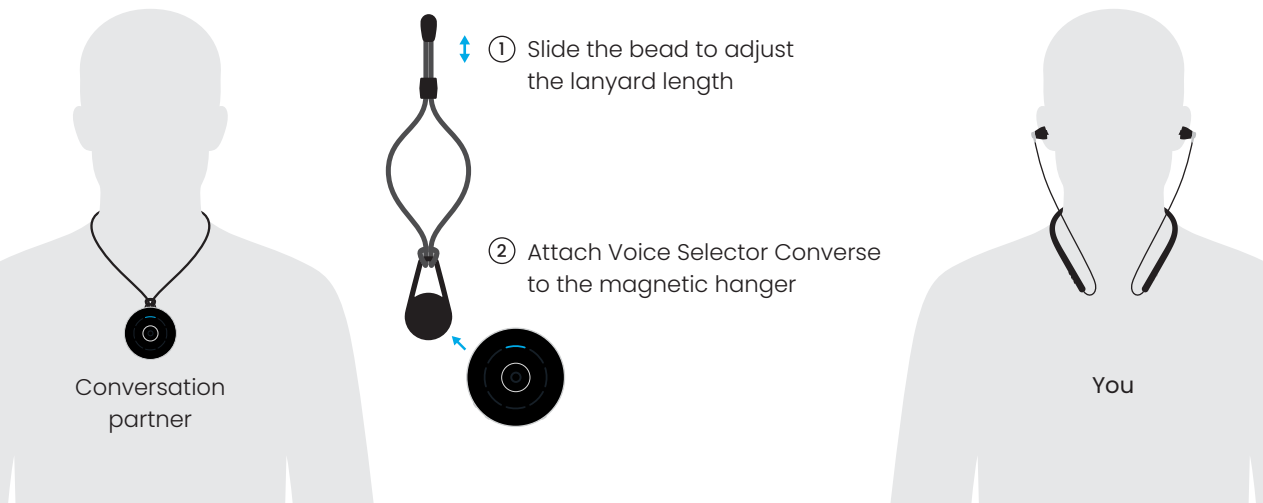

Volume adjustment


For optimal performance, set volume level so voices sound natural. High volume may amplify undesired voices and background noise.

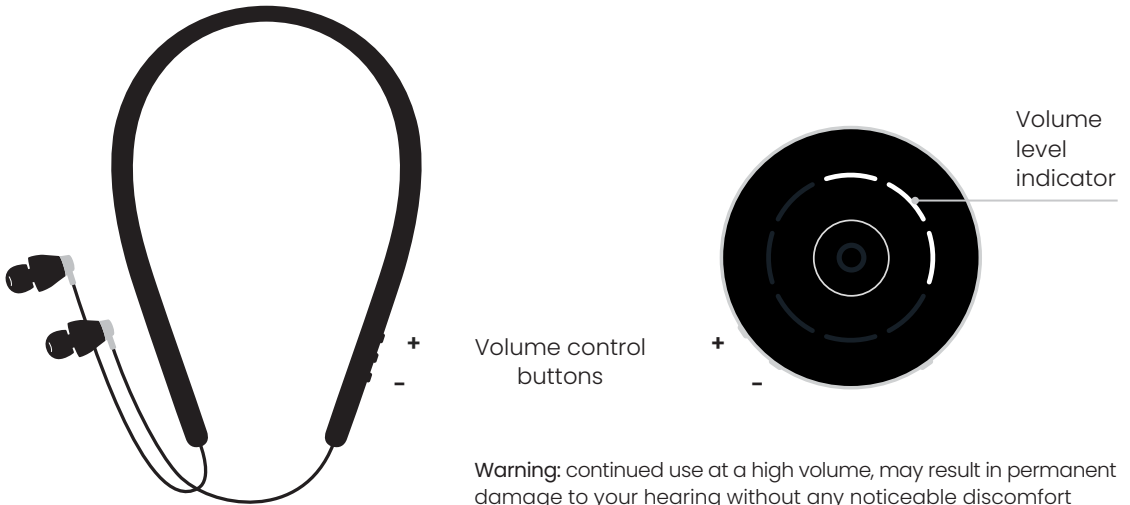

Warning: continued use at a high volume, may result in permanent damage to your hearing without any noticeable discomfort

Charging both devices

Use the Micro-USB cable to charge your Voice Selector Converse and the earphones

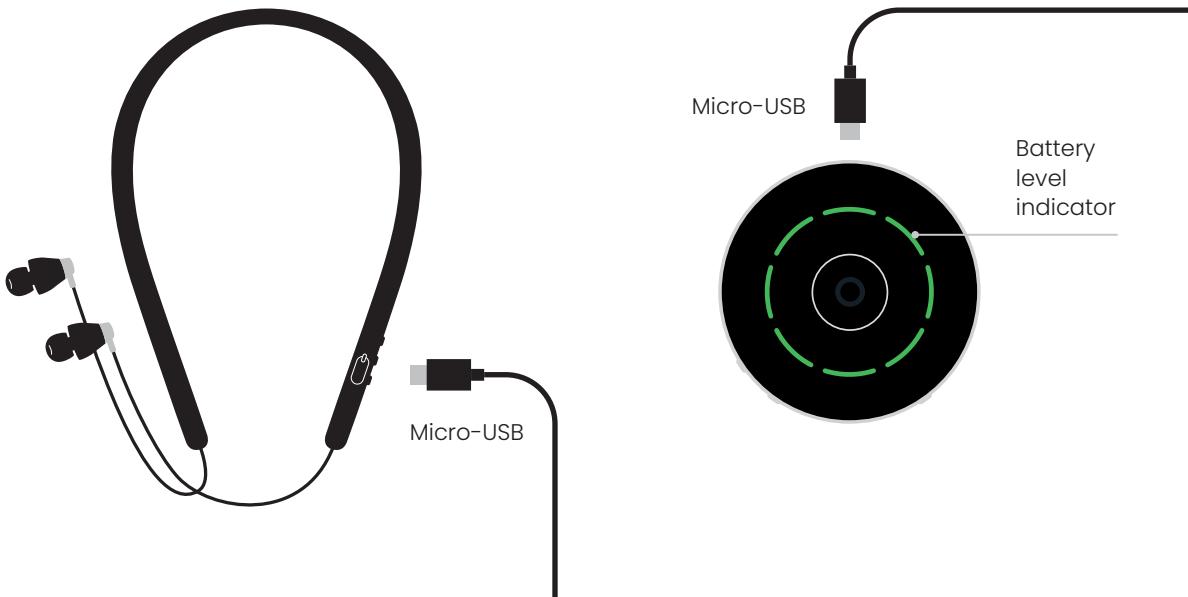


Earphone fitting

Choose the most comfortable fit for your ears.

A good seal will reduce outside noise and improve speech understanding.

To remove tip, pull firmly

To attach tip, push until it clicks

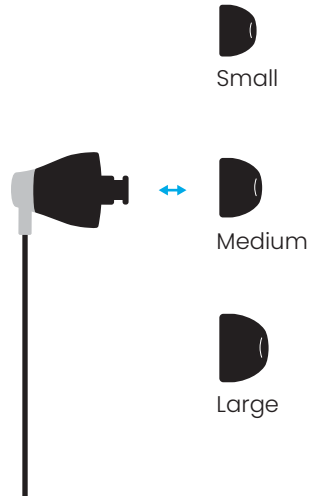


Nuance Hearing app


Download the app to unlock additional features



Scan the QR code

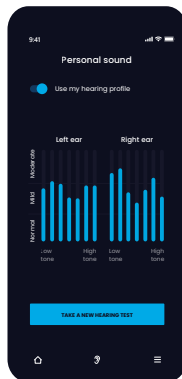

Take a hearing test to create a personal hearing profile

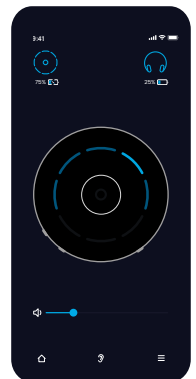


Use the app as a remote control for the device

Earphones Bluetooth pairing

To pair the earphones to any Bluetooth device or with Voice Selector Converse, follow these steps:

① Set the earphones in pairing mode

- Press the power button to turn off the earphones.
You'll hear: "Power off"
- Wear the earphones and **long-press** the power button.
You'll hear: "Power on... Pairing"

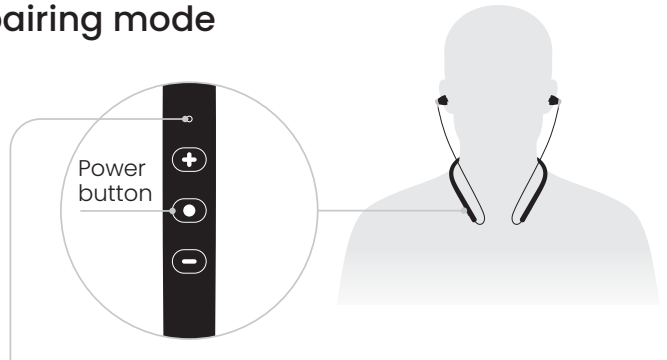

Indications: • • Flashing red and blue • Blue
 Searching for a new device to pair with Pairing complete

②^A Pair with smart TV, smartphone or PC


- Open the Bluetooth settings of the device you'd like to pair with
- Select the name of your earphones: BEP240C
You'll hear: "Pairing successful, connected"



②^B Re-pair with Voice Selector Converse

Only when the pairing is lost

- Remove the silicone case from the device
- **Long-press** the Bluetooth button until it enters pairing mode

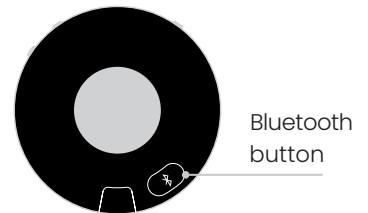

Indications: No paired earphones Pairing mode Pairing complete



Safety Instructions

How to use the device

Voice Selector Converse including wireless earphones is a unique assistive listening device, reducing ambient noise and focusing audio in the selected direction.

Storage and operation temperatures should be between 10° and 45°C, and humidity 10% to 95%.

How to clean the device

The device should be cleaned with a damp cloth.

Instructions for disposing of electronic equipment:

The electronic waste must be treated according to local regulations. Nuance Hearing Ltd. recommends recycling the device at the regional disposal of electronic equipment. Local laws must be followed.

According to the regulations, we are required to explain the following warnings:

- Warning: Prolonged listening at high volume is not recommended and may cause irreversible damage to your hearing
- Warning: Use of accessories, components, or spare parts not supplied by Nuance Hearing may impact use and void the warranty
- The battery in Voice Selector Converse and in the wireless earphones are not replaceable and should not be replaced
- If you use other medical devices or wear implantable medical devices such as defibrillators or pacemakers and are concerned that your Voice Selector Converse might cause interference with your medical device, please contact your physician or the manufacturer of your medical device for information about the risk of disturbance.

Standards and regulations

Voice Selector Converse and wireless earphones meet the strictest international standards for electrical safety and electromagnetic compatibility.





We're here to help

Scan the QR code for online support
support@nuancehear.com
www.nuancehear.com