


User guide




Getting started

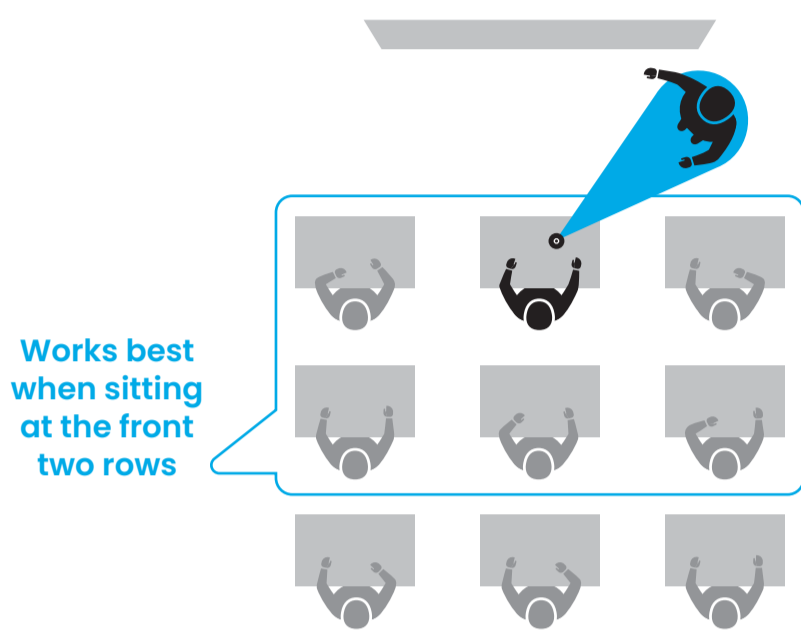

Switch between the 3 modes described in the following pages:

Lecture


Automatically track the voice of the teacher who moves in the front of the classroom

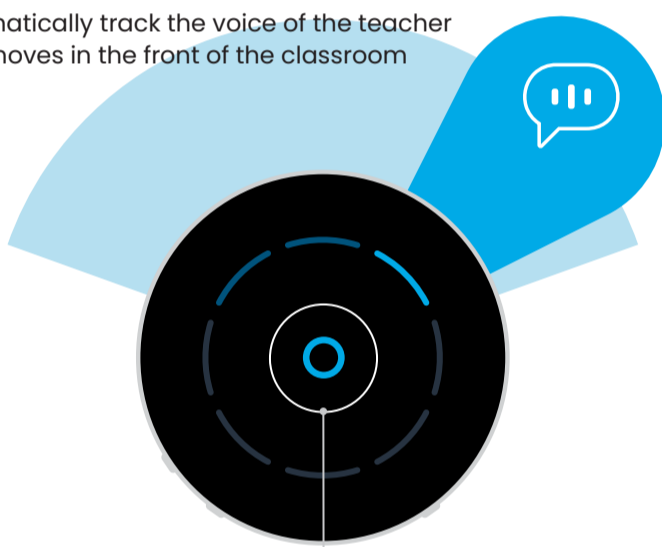

Tap the center button to switch from Lecture to Discussion mode

Discussion


Automatically track the dominant speaker, wherever they are in the room

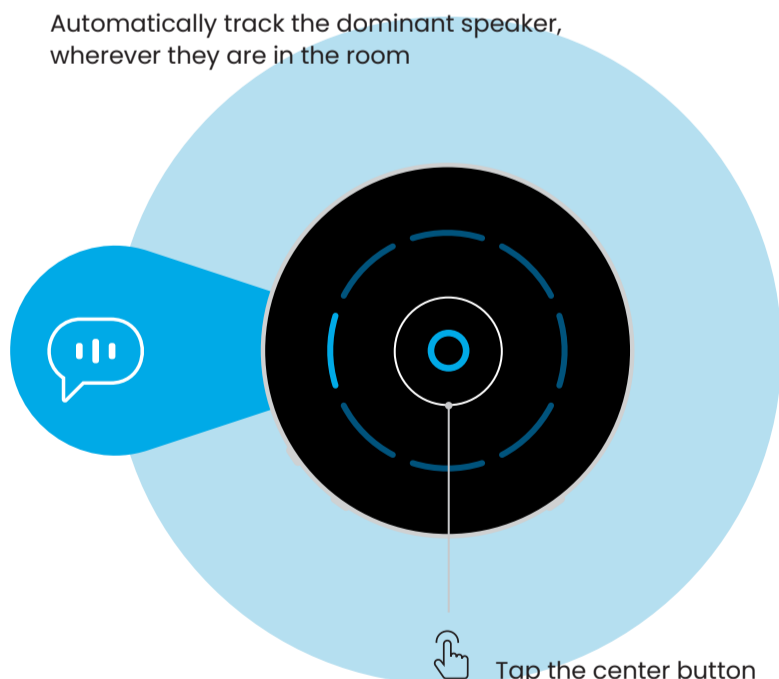

Tap the center button to switch from Discussion to Lecture mode


Conversation

Manually select the direction of your conversation partner

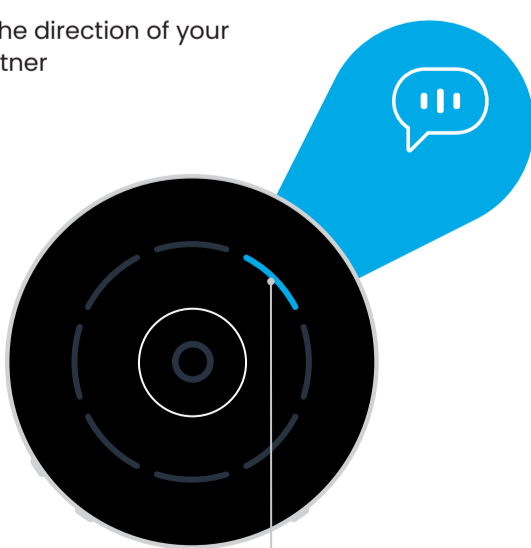


Tap the segment in the direction of a particular speaker

Battery and charging

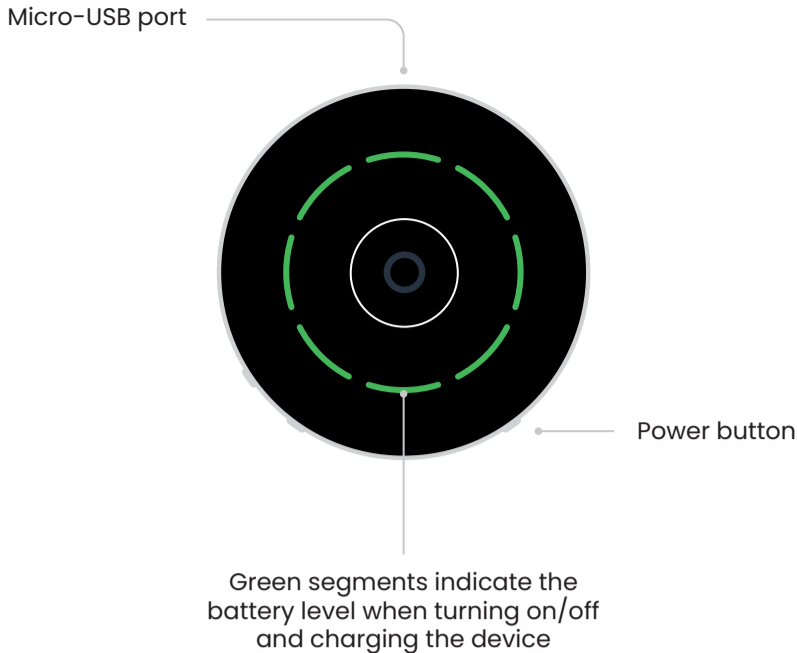

Volume


For optimal performance, set the volume so voices will sound natural. Higher volume may amplify undesired voices and background noise.

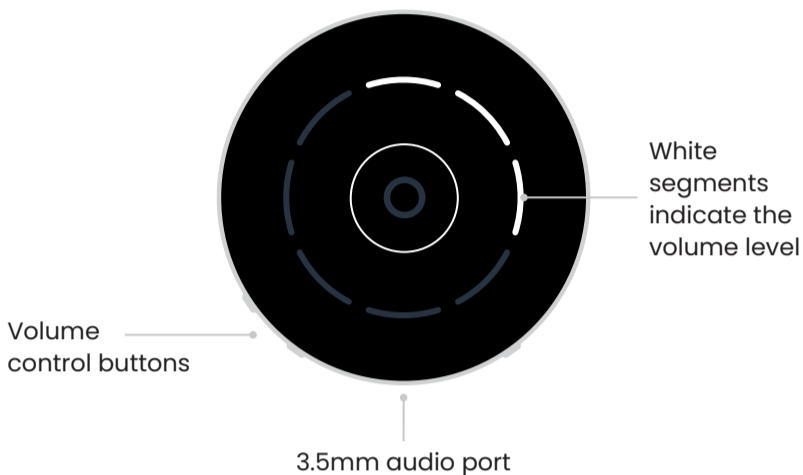

Warning: with continued use at a high volume, your ears can become accustomed to the sound level, which may result in permanent damage to your hearing without any noticeable discomfort.

Earphones

You can use any earphones with a 3.5mm audio plug

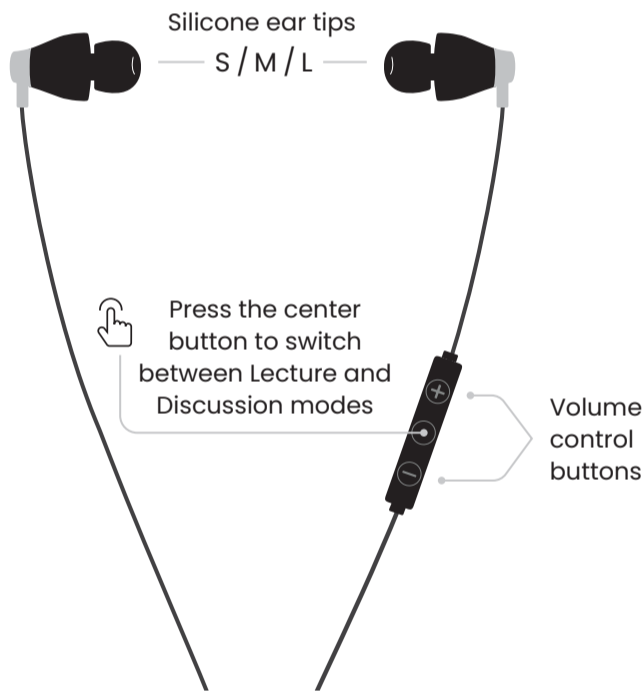

Safety Instructions

How to use the device:

Voice Selector Study, including wired earphones, is a unique assistive listening device, reducing ambient noise and focusing audio in the selected direction. Storage and operation temperatures should be between 10° and 45°C, and humidity 10% to 95%.

How to clean the device:

The device should be cleaned with a damp cloth.

Instructions for disposing of electronic equipment:

The electronic waste must be treated according to local regulations. Nuance Hearing Ltd. recommends recycling the device at the regional disposal of electronic equipment. Local laws must be followed.

According to the regulations, we are required to explain the following warnings:

- Warning: Prolonged listening at high volume is not recommended and may cause irreversible damage to your hearing
- Warning: Use of accessories, components, or spare parts not supplied by Nuance Hearing may impact use and void the warranty
- The battery in Voice Selector Study is not replaceable and should not be replaced
- If you use other medical devices or wear implantable medical devices such as defibrillators or pacemakers and are concerned that your Voice Selector Converse might cause interference with your medical device, please contact your physician or the manufacturer of your medical device for information about the risk of disturbance.

Standards and regulations:

Voice Selector Study meets the strictest international standards for electrical safety and electromagnetic compatibility.



We're here to help



Scan the QR code for online support
support@nuancehear.com
www.nuancehear.com